

# Ultra-micro bubbles skin beauty instrument

## Manual (Universal Edition)

**Dear user:**

Thank you for purchasing this product!  
Please read this manual carefully before use this  
product  
to ensure the best performance and safety of it.





## Service and Repair

1. The instrument from the date of purchase (with bills) host one year warranty, spare parts and consumables for non-warranty. In the non-warranty coverage or warranty period, but also responsible for repair and update (to recover the cost of the fee).

2. Accessories and consumables, for replenishment or purchase, please contact your dealer.

3. The following human factors caused the failure, does not provide free warranty service.

(1) Remove, modify equipment due to malfunction.

(2) Improper use, the lack of reasonable maintenance caused by failure.

(3) Inadvertently beat, drop caused by the failure.

(4) Failure to follow the correct instructions for use has resulted in malfunction

## Principle and Efficacy

### 2.Efficacy



Effective in removing acne, seborrheic alopecia, folliculitis, and allergen.



Whiten and refine skin, remove hyperpigmentation.



Deeply clean skin, moisturize skin, promote skin metabolism.



Remove blackheads, shrink pores, smooth skin.



Shape and tighten chin skin, shrink pores, prevent double chin.



Provide skin care before or after treatment of Erythroderma, anti-aging.


## Accessories Show (Internal accessories can not be lost)

### A Bubble releasing skin cleaning pen (with eight water sculptures):

First use a bubble pen to clean the skin, sucking from the middle to both sides ,rapid sliding,do not stay too long.Sensitive muscle with a flat and a vortex head,large pores,T-zone matte surface with suction head.

### B Micro electrical therapy head:

BIO facial lifting,improve cell permeability,help the body to better absorb nutrients and water,oxygenated blood and so on,and increase collagen synthesis,reduce wrinkles,make skin softer,more elastic and shiny.

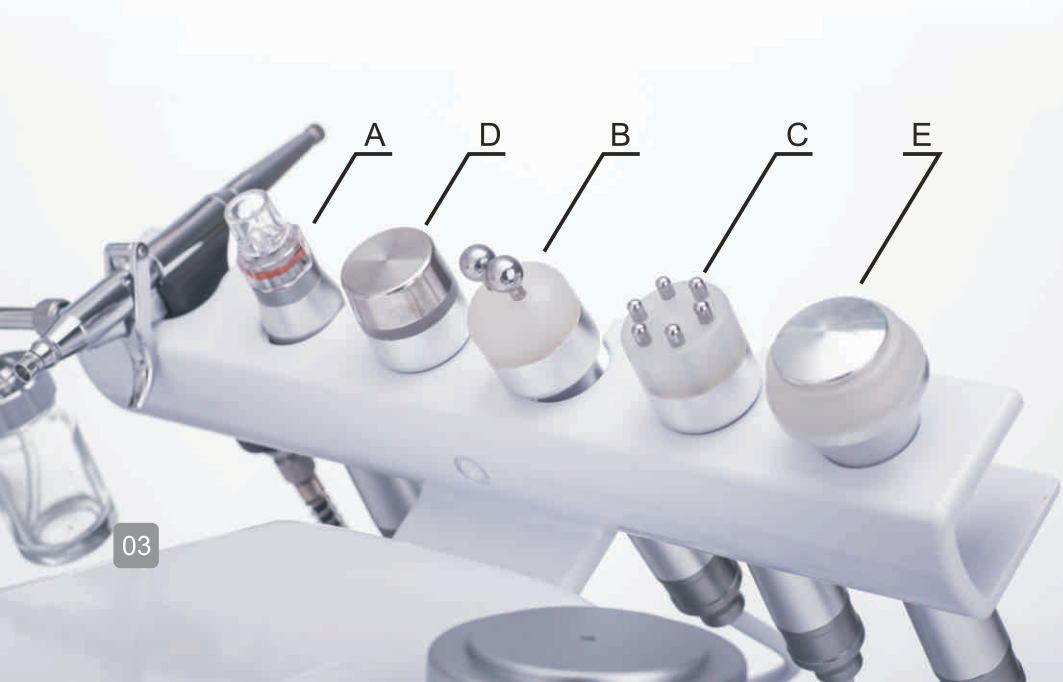

## Packing List

Description	Qty.	Remarks
Main machine	1pcs	
Power cable	1pcs	
Bubble releasing skin cleaning pen	1pcs	With eight water sculptures
Micro electrical therapy head	1pcs	
Ultrasonic head	1pcs	
RF radio head	1pcs	
Icy skin-repairing treatment head	1pcs	
Fuse	2pcs	3A,for spare
Instructions	1pcs	

### Note:

1. Please prevail in kind, if there is a special need, please contact the dealer directly
2. Non-native accessories, do not connect the machine to use, so as to avoid injury.

## Simple Troubleshooting

Fault status	Method of check and repair
Open the switch, the host no response	Check if the power is connected correctly
No output	Check if the attachment is properly connected to the host
In use, the fuselage overheated	Use time is too long, turn off the natural cooling
	Check the voltage is normal (to be professionals)
The entire machine is not powered	Check the fuse is burned out, if burned, please replace

## Technical Parameters

**Voltage Input:** AC 110V~220V

**Frequency:** 50Hz

**Working Power:** 26W

**Carton Size:** 50\*41\*37CM

**Note:** The design and description of this product are subject to change and update without prior notice

## Accessories Show

**(Internal accessories can not be lost)**

### C RF radio head:

Open the pores: Positioning tissue heating, open the pores to promote further contraction of the skin taut, relieve wrinkles, produce new collagen, skin will be more compact later.

### D Ultrasonic head:

Ultrasonic import products deep into the subcutaneous tissue, promote micro circulation, activate and brighten the skin, can be imported into the skin care products to penetrate the skin to accelerate absorption.

### E Icy skin-repairing treatment head:

Frozen skin, shrink pores, release the power of ice, the skin penetrates deep into the skin, instantly promote strong contraction of the pores of the skin, the skin surface to form a natural cold film to lock in nutrients

## Accessories Show

Eight water sculptures (Accessories of Bubble releasing skin cleaning pen), suitable for different parts.

### Four kinds of water suction method:

Flat,  
straight,  
focus and vortex for different types of skin.  
Thorough clear, from



① Flat absorption head:  
water sculpture head is of flat shape.



② Straight head:  
inner of water sculpture is of straight shape.



③ Focus head:  
inner of water sculpture is of point by point shape.



④ Vortex head:  
inner of water sculpture is of point vortex head.


## Tips



Three liquid containers,  
counterclockwise rotation,  
removed from the base.

### Note:

It is not recommended to  
unscrew the container from the  
base, so as to avoid wear and  
tear, affecting the sealing of the  
container.

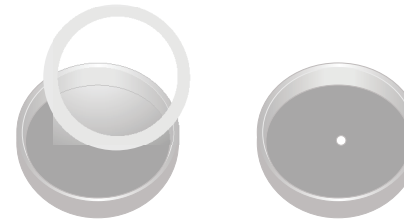

**Note:** There is a small hole in the center of the sewage  
container lid and no seal inside.  
Inside the container lid, the silicone seal on the container  
base should not be lost.

## Tips




Gather the container of clean water after using the bubble pen (hole in the center of the lid to discharge negative pressure gas).

Contains clean liquid container.

Liquid flow adjustment adjustment knob (the more counterclockwise rotation, the more liquid into the bubble pen).




Bubble pen fixed to the back of the fuselage, can not be removed.

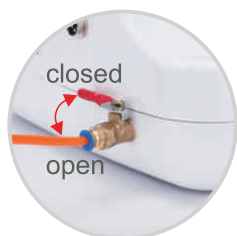

Waste water valve, the valve is closed drainage.

Turn the valve 90 degrees counterclockwise to close the valve

## Touch Screen and Port Icon

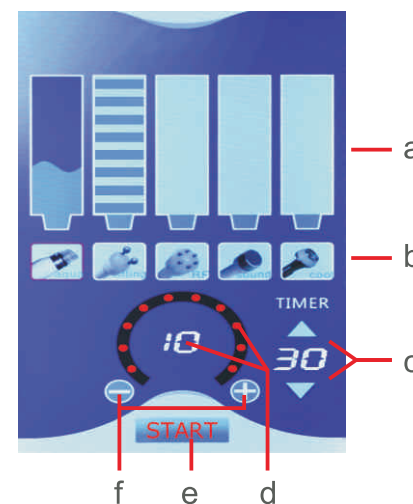

a. Accessory output progress bar(the left most bar indicates the bubble releasing pen's ON/OFF status)

b. Accessory selecting(The selected accessory icon has a red border)

c. Time adjustment and display



d. Out put intensity of accessories.

e. Switch on(red)/pause(white)

f. Intensity adjustment for selected accessory(In addition to bubble pens can not be adjusted, the other are 0 to 10 files can be adjusted)

## Touch Screen and Port Icon

- A. Icy skin-repairing treatment head
- B. Bubble releasing skin cleaning pen(This accessory can not be removed)
- C. RF radio head
- D. Micro electrical therapy head
- E. Ultrasonic head
- F. Waste water valve(The sample picture is closed)

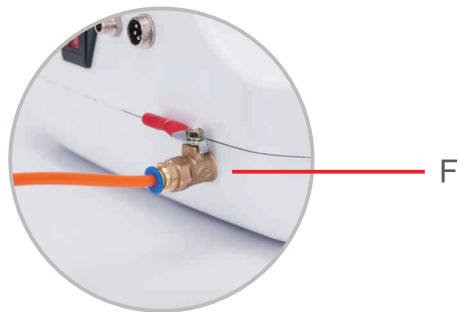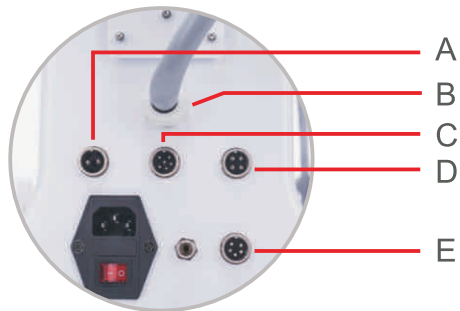

## Steps

- 1.Connect the accessories which will be used,
- 2.Turn on the power, open the main power switch, the instrument into standby mode.
- 3.With your finger click the icon at (b) and select Accessories. From left to right in order Bubble releasing skin cleaning pen, Micro electrical therapy head,RF radio head,Ultrasonic head, Icy skin-repairing treatment head.The selected accessory icon has a red border
- 4.With your finger click at (f)"-,+”to adjust the selected accessory output strength.  
(0 ~ 10 files can be adjusted, 0 is no output, bubble pen can not adjust the intensity)
- 5.With your finger click at (c) up and down arrows,adjust working hours (1 ~ 60minutes adjustable, the default boot time is 30 minutes)
- 6.With your finger click at (e)"START".The text red is running, the white is paused.
- 7.Set the relevant output, use the appropriate accessories for care.
- 8.After care, please remove the accessories, clean and dry, placed in a dry and ventilated place for the next use.